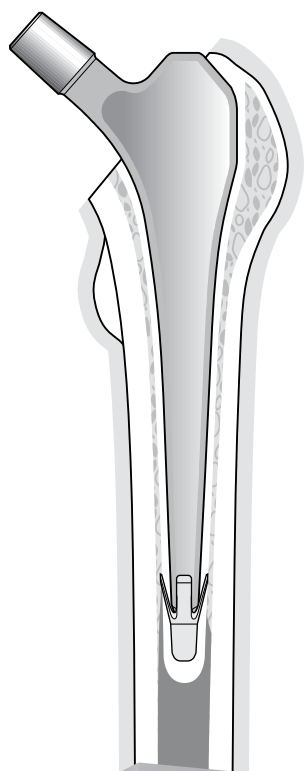
S-TAPER MIRROR

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OPERATING TECHNIQUE

- Modular system with distal centralizer
- Mirror surface finishing
- Long stem with wedge for lateral fractures

S-TAPER MIRROR

ENDOPROSTHETIC SYSTEM FOR THE TROCHANTERIC REGION OF THE OLD PATIENT

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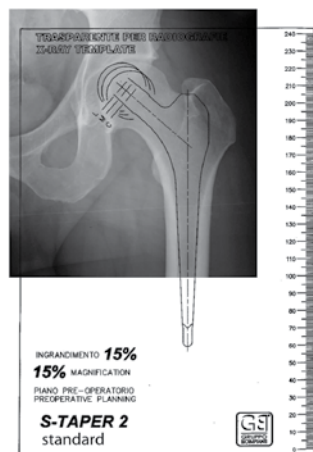
PRE-OPERATIVE EVALUATION (standard and long)


Pre-operative evaluation of the size and position of the femur stem in the antero-posterior and lateral plane enable the surgeon to adopt the correct approach to the operation. A bilateral X-ray of the pelvis will help gauge the length of the limb, which has to be determined pre-operatively and then corrected. The length can be corrected by using the femur heads in five different lengths. The pre-operative templates of the "S-TAPER MIRROR" stem are available in a 15% enlarged format.

Use of an X-ray reference may help determine the enlargement of the patient's X-ray. Align the template along the anatomical axis of the femur keeping the stem in the femoral channel and the centre of rotation of the femur head correctly positioned. **Choosing a stem by pre-operative evaluation is an approximate method. The final size of the stem will be determined intra-operatively.**

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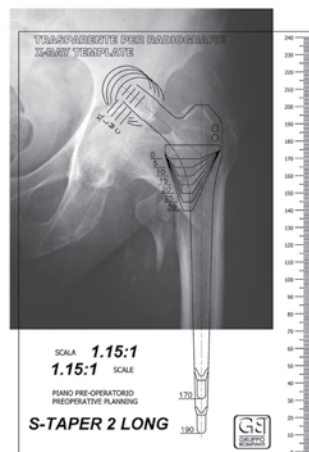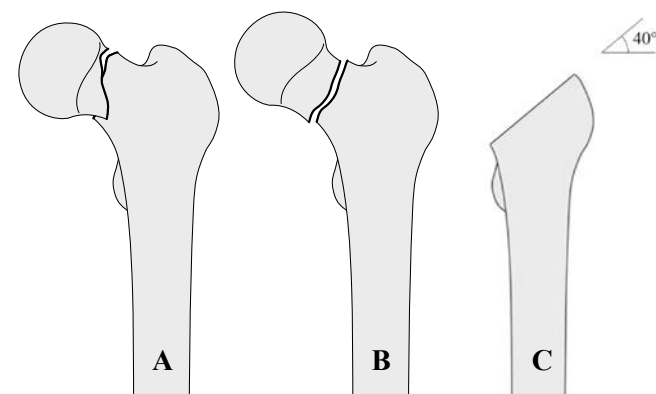
OSTEOTOMY OF THE FEMUR NECK (standard)

Having freed the femur neck, proceed with osteotomy by resection as laterally as possible at the transition of the great trochanter. This is done at an angle of around 40° and around 15 mm above the little trochanter (fig. C standard).

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
INDICATIONS (long)

The long stem with wedge is used for lateral fractures of the femur neck.

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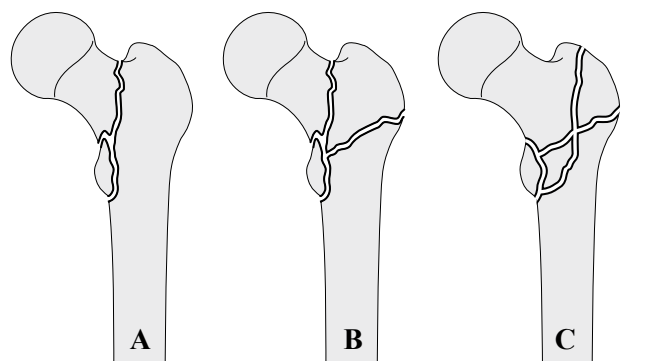PREPARATION OF THE FEMORAL CANAL (standard and long)

Before proceeding with the broaches, the medial part of the great trochanter can be removed by using the Great Trochanter Osteotome (11-0381010). Removal of this part of the bone facilitates insertion of the broaches in the axial position.

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PREPARATION OF THE FEMORAL CANAL (standard and long)

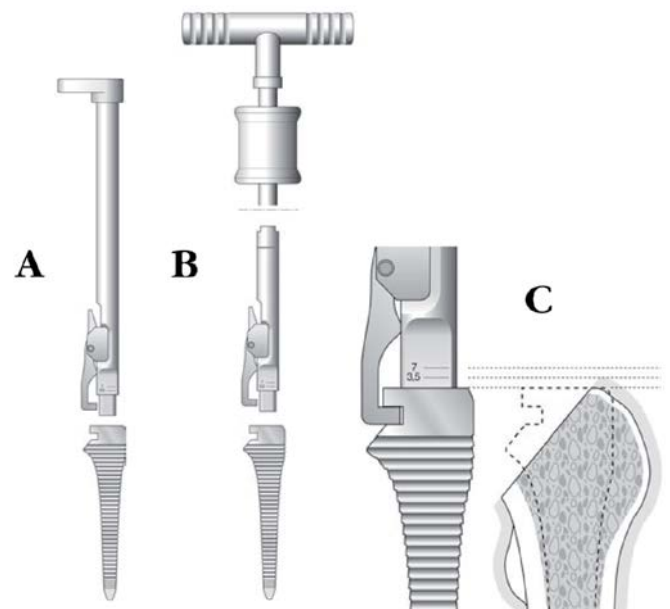
Once the femoral canal has been opened up with the great trochanter osteotome, mount the Broach Handle (12-0380123) on the Broach (12-03800**) or the Broach Sliding Hammer (12-0380116) to the Union with Hook for Broach (12-0380117) and then to the broach. The notches on the broach handle indicate which prosthetic head to use. Once the definitive broach has stopped, check the joint between broach and handle by the apex of the great trochanter; if it's flush, the head will be medium, if it's by the 3.5 notch will be long and if by the 7 notch it will be extra long.

A Option 1: without sliding hammer

B Option 2: with sliding hammer

C Detail of notches for determining the neck of the femur head

Standard - Long

S-TAPER MIRROR

ENDOPROSTHETIC SYSTEM FOR THE TROCHANTERIC REGION OF THE OLD PATIENT

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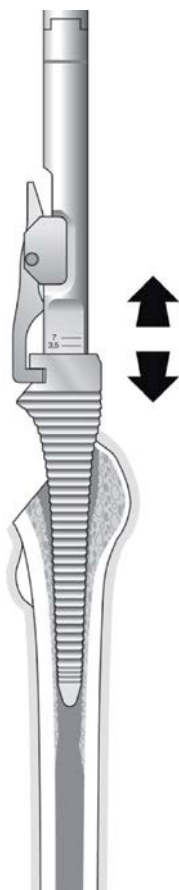
BROACHING THE FEMORAL CANAL (standard and long)

Correct insertion of the first broach, which must be lateral to the great trochanter and axial to the inside of the femoral canal, is crucial, as it prevents the final implant twisting inwards.

The first broach determines the positioning of all the others. When the broach proceeds no further along the femoral canal, the size used will be the size of the stem.

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TEST REDUCTION (standard)

Once the broach has been determined, the last broach used is left "in situ" and the test reduction can be started. In the hole on the inclined plane of the broach, insert the Test Head Adaptor (12-0380125) and then the test heads (11-03809**) of four different lengths. Proceed with the reduction by joining the broach+test head to the previously implanted cup and checking the length of the leg and the tension of the soft tissues.

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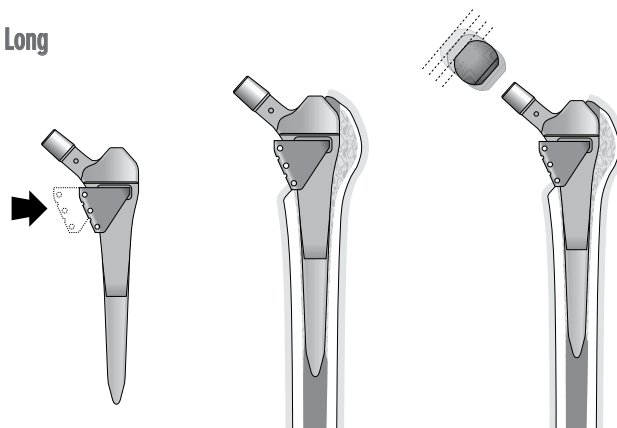
TEST REDUCTION (long)

After preparing the femoral canal, the test wedge (12-03813**) is mounted on the test stem (12-03816**) and fed into the femoral canal. The test heads (11-03808**) are then assembled in four different lengths. Proceed with reduction by joining the test stem to the previously implanted cotyle or cup and checking leg length and soft tissue tension.

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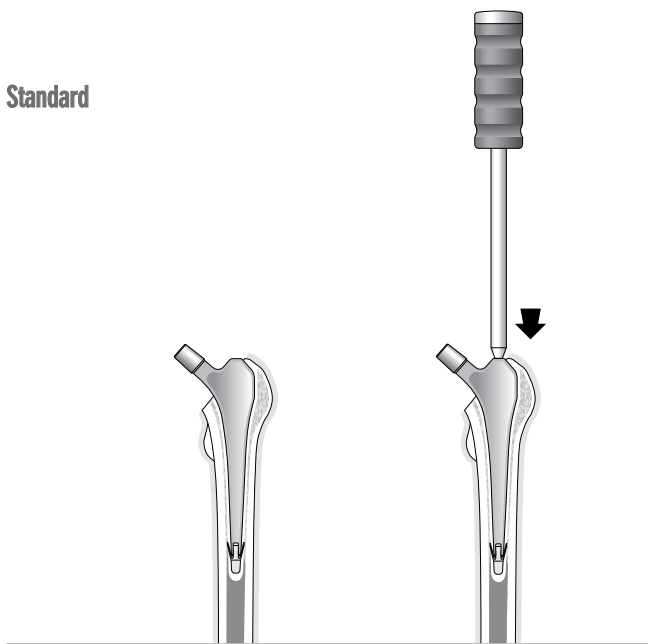
STEM INSERTION (standard and long)

After broaching the femur, the femur canal plug (Esoblock) is inserted and cemented, preferably with Bioceium or Biogent (with antibiotic). The distal tip is applied to the stem and the whole inserted manually. Use Impactor (12-0380122) inserted in the proximal hole of the stem.

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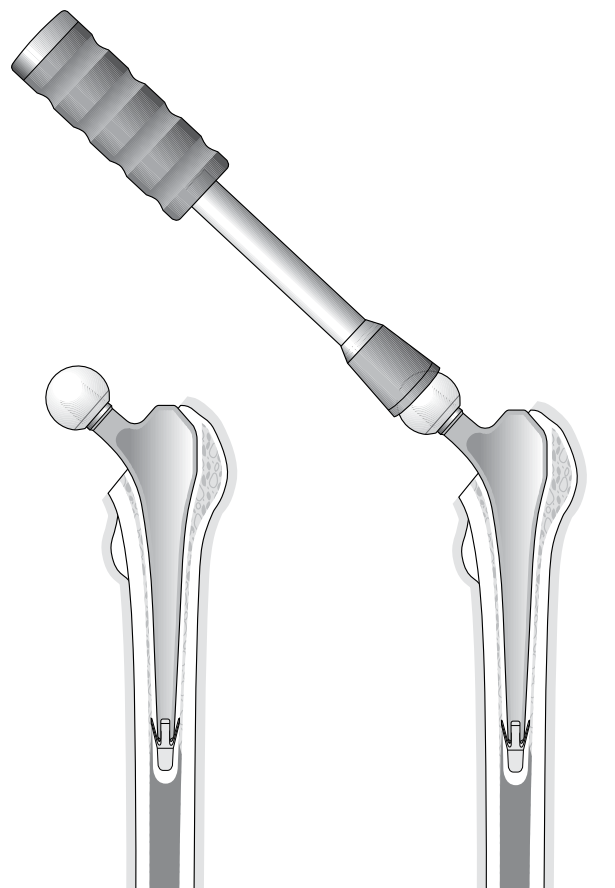
POSITIONING THE FEMUR HEAD (standard and long)

Having inserted the stem, it is advisable to repeat the test reduction with the test heads mentioned in step 6 above and then assemble the definitive head chosen. If it's ceramic, insert it manually and make a half clockwise turn; if metal, use the Head Impactor (11-0380800).

Standard - Long

S-TAPER MIRROR

ENDOPROSTHETIC SYSTEM FOR THE TROCHANTERIC REGION OF THE OLD PATIENT

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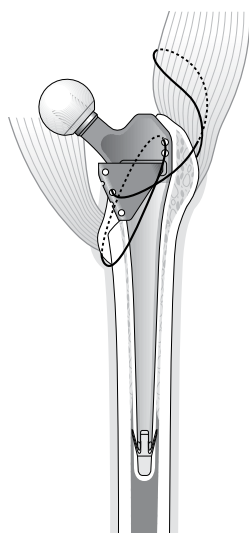
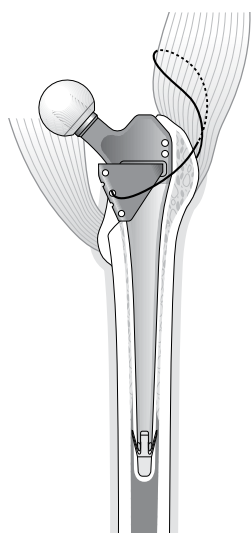
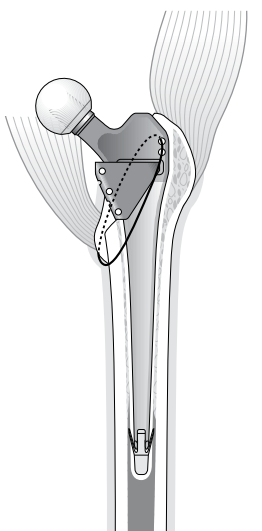
CERCLAGES (only for long)

The stem and wedges have holes to accommodate cerclage wires between stem and lesser trochanter, wedge and greater trochanter or both.

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

STEM REMOVAL (standard and long)

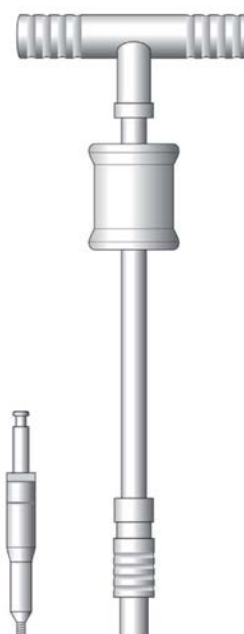If the stem has to be removed, use the Extraction Union (12-0380115) which fits onto the hammer. The extraction union is screwed into the proximal threaded hole in the stem.

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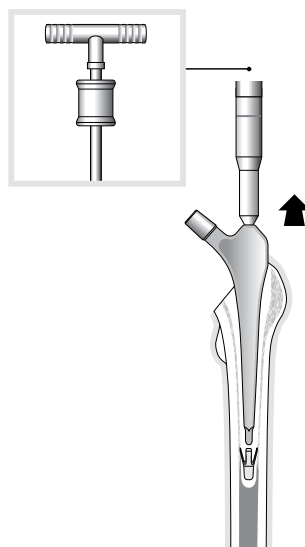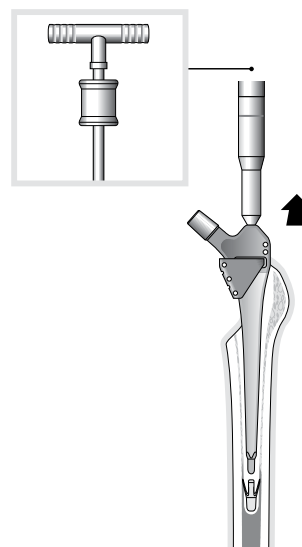
All components of the implant must be handled and stored with extreme care, as any alteration (deformation, scratches, and so on) could put the implant at risk. In the operating theatre it is likewise necessary to make sure the stem isn't impaired by the instruments.

For reasons of compatibility between materials and geometrical form, only GRUPPO BIOIMPIANTI instruments should be used. As with implant materials, great care must also be taken with the instruments, which must be stored in their racks to prevent accidental impact and surface damage.

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Each stem is supplied appropriately packaged and marked ensuring error-free identification and sterilized with B rays.

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9 TESTINE DI PROVA 28 MM
28 MM TRIAL HEADS

REF.	LENGTH
11-0380860	short
11-0380870	medium
11-0380880	long
11-0380890	extralong

10 TESTINE DI PROVA 32 MM 32 MM TRIAL HEADS

REF.	LENGTH
11-0380960	short
11-0380970	medium
11-0380980	long
11-0380990	extralong

11 TESTINE DI PROVA 36 MM 36 MM TRIAL HEADS

REF.	LENGTH
11-0381060	short
11-0381070	medium
11-0381080	long
11-0381090	extralong

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GRUPPO BIOIMPIANTI S.r.l.