“BASIC-NAIL”

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The locked nail “Basic-Nail” for femoral pertrochanteric fractures (standard or medium version) is an alternative to open synthesis techniques. It provides reduction and immobilization of the fracture by a closed technique which limits the risk of bone infection and blood loss.

INDICATIONS AND CONTRAINDICATIONS

Choosing this type of implant and size to use is a matter of general traumatological principles and of the surgeon’s experience. Generally speaking, the “Basic-Nail” is used for:

- 1) **basicervical fractures;**
- 2) **perthrochanteric fractures;**
- 3) **intertrochanteric fractures;**
- 4) **subtrochanteric fractures**

If any of these is accompanied by a diaphyseal fracture, the “Basic-Nail” medium are used.

The implant should not be used when there is:

- **inadequate quantity and quality of bone tissue;**
- **infection in progress;**
- **excessive geometrical anomaly in the shape of the femur**

As with all osteosynthesis systems and in spite of correct sizing, there is a risk of fatigue-induced breakage in the event of delayed union, non union or movement following removal of the distal screws.

ADVANTAGES

Closed osteosynthesis using an endo-medullary nail of variable length brings several advantages:

- **versatility;**
- **speed;**
- **facilitated osteosynthesis of minimum aggressiveness;**
- **maximum stability;**
- **rapid patient recovery**

Furthermore, thanks to the special characteristics of the “Basic-Nail” with respect to other nails on the market, the “Basic-Nail” system also has following benefits:

- **easy and rapid implant thanks to the low valgus angle (4°);**
- **removal is facilitated because of the anodization type II;**
- **inter-fragmentary compression on the fracture line;**
- **nail inserter for the distal holes aiming for standard and intermediate (250 ml) nail;**
- **possibility to use a Steinman wire to avoid any rotation during lag screw insertion;**
- **tapered lag screw, make easier the introduction and save Bone Stock.**

MATERIALS

The nail “Basic-Nail” is made of titanium alloy Ti 6Al 4V grade 5 ELI in accordance with ISO 5832/3. Surface treatment with anodization type II DOTIZE®.

DOTIZE® is a registered trade name of DOT gmbh.

COMPONENTE DIAFISARIA ENDOMIDOLLARE STANDARD
DIAPHYSEAL ENDO-MEDULLARY COMPONENT, STANDARD

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*Length 195 mm, distal Ø 9 or 10 mm. Designed for
 transcervical component angles of 125° and 130°.*

Codice Ref.	Angolo CCD CCD Angle	Ø mm distale distal Ø mm
14-0480925	125°	9
14-0481025	125°	10
14-0490930	130°	9
14-0491030	130°	10

COMPONENTE DIAFISARIA ENDOMIDOLLARE INTERMEDIA L. 250 mm
DIAPHYSEAL ENDO-MEDULLARY COMPONENT, INTERMEDIATE 250 mm

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*Length 250 mm, distal Ø 9 or 10 mm. Designed for
 transcervical component angles of 125° and 130°.*

Codice Ref.	Angolo CCD CCD Angle	Ø mm distale distal Ø mm
14-0614125	125°	9
14-0615125	125°	10
14-0614130	130°	9
14-0615130	130°	10

COMPONENTE TRANSCERVICALE VITE LAG CONICA
TRANSCERVICAL SCREW COMPONENT TAPERED LAG SCREW

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*11 different lengths from 85 to 120 mm, in 5 mm steps.
 Self-tapping and self-drilling.*

Codice Ref.	Lunghezza mm Length mm
14-0402085	85
14-0402090	90
14-0402095	95
14-0402100	100
14-0402105	105
14-0402110	110
14-0402115	115
14-0402120	120

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Different lengths from 30 to 80 mm, in 5 mm steps. Self-tapping

Codice Ref.	Lunghezza mm Length mm
14-0400230	30
14-0400235	35
14-0400240	40
14-0400245	45
14-0400250	50
14-0400255	55
14-0400260	60
14-0400265*	65
14-0400270*	70
14-0400275*	75
14-0400280*	80

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STRUMENTARIO - *INSTRUMENTATION*

TELAI PORTA STRUMENTI - *INSTRUMENT TRAYS*

Base "Basic-Nail" - *Container "Basic-Nail"*

Codice - Ref.	14-0040518
----------------------	------------

Inserto "Basic-Nail" - *Insert "Basic-Nail"*

Codice - Ref.	14-0040519
----------------------	------------

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GRUPPO BIOIMPIANTI S.r.l.