



# K-MOD SKINNY

## TECHNICAL DETAILS

SKINNY FEMORAL COMPONENTS are designed to be implanted when the AP dimension is appropriate, but coverage on ML plan is excessive.

Skinny components, also known as «Narrow», are characterized by a lower coverage on ML plan, for this reason they address mainly female patients.

Skinny's measures are indicated by the symbol – (2-, 3-, 4-) and were designed from Standard femur measures 2, 3 and 4.

- Dimension in AP equal to the reference size.
- Indicate for female patients.
- Congruence: equal to the reference size, as reported by the color code on the package.
- Dimension in ML equal to that of one size smaller, equivalent to a symmetrical reduction of about 2mm on both sides.

**K-MOD**  
KNEE-MODULAR SOLUTION



*Enjoy Mobility*

## MATERIALS

SKINNY femoral components are manufactured in Cobalt Chrome Molybdenum (CoCrMo) alloy, according to the ISO 5832-4.

The uncemented component is provided with a Ti-Growth-C® titanium plasma spray coating, whereas the allergy version with a titanium-niobium nitride coating.

Each component is provided appropriately packaged and marked ensuring error-free identification and sterilized with B rays.

## INSTRUMENTATION

The introduction of the SKINNY Femoral Components DO NOT require additional elements or steps in instrumentation, in particular:

- 4in1 Cut Block equal to the reference size.
- Trial Reduction: Skinny Femur Trials are not provided, therefore use those of the reference size. In this phase, evaluate coverage on ML: if it is excessive, proceed with the preparation as usual and subsequently implant the SKINNY version.


## SIZES

SKINNY FEMUR	AP COVERAGE	ML COVERAGE	4in1 CUT BLOCK	FEMUR TRIAL	PE INSERT
2-	Fem 2	Fem 1	Size 2	Fem 2	Size 2
3-	Fem 3	Fem 2	Size 3	Fem 3	Size 3
4-	Fem 4	Fem 3	Size 4	Fem 4	Size 4

Reference size chart of Skinny Femoral Components

FEMUR	ML (mm)	AP (mm)
2-	59	58
2	63	58
3-	63	62
3	67	62
4-	67	66
4	71	66

Comparison of Standard to Skinny sizes

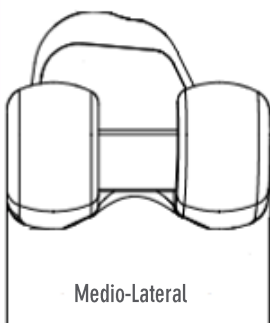

## CODES

### CEMENTED AND ANALLERGY SKINNY FEMORAL COMPONENT

CEMENTED	DESCRIPTION	ANALLERGY
120701112	K-MOD SKINNY Cemented femur 2- Left	120701112A
120701113	K-MOD SKINNY Cemented femur 3- Left	120701113A
120701114	K-MOD SKINNY Cemented femur 4- Left	120701114A
120701212	K-MOD SKINNY Cemented femur 2- Right	120701212A
120701213	K-MOD SKINNY Cemented femur 3- Right	120701213A
120701214	K-MOD SKINNY Cemented femur 4- Right	120701214A

### UNCEMENTED SKINNY FEMORAL COMPONENT

UNCEMENTED	DESCRIPTION
120700112	K-MOD SKINNY Uncemented femur 2- Left
120700113	K-MOD SKINNY Uncemented femur 3- Left
120700114	K-MOD SKINNY Uncemented femur 4- Left
120700212	K-MOD SKINNY Uncemented femur 2- Right
120700213	K-MOD SKINNY Uncemented femur 2- Right
120700214	K-MOD SKINNY Uncemented femur 2- Right

